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RhinoBlock hay fever nasal ointment

Product description

RhinoBlock is an easy-to-apply nasal ointment based on pure, white vaseline and pure honey. RhinoBlock hay fever nasal ointment creates a protective layer in the nose against grass and tree pollen, house-dust mites and other irritants. This protective layer relieves the allergic symptoms caused by pollen and house-dust mites.

Ingredients

Petrolatum, Mel.

Indications

RhinoBlock can be used to relieve allergic symptoms caused by pollen and house-dust mites.

Instructions for use

Apply RhinoBlock in both nostrils, 2 - 4 times a day or more frequently, as required. Apply with a cotton bud or similar. Then spread the nasal ointment inside your nose by lightly massaging the outside of your nostrils. To protect against airborne pollen, apply RhinoBlock before going outdoors for a long period of time (e.g. to exercise, walk, cycle or garden). For protection against house-dust mites, apply RhinoBlock nasal ointment at least once before going to bed.

Use during pregnancy and breastfeeding

There are no indications that RhinoBlock cannot be used during pregnancy and breast feeding.

Do not use

- Do not use if there is hypersensitivity to any of the ingredients.
- Do not use in combination with oxygen therapy, e.g. when using an oxygen cannula. Petrolatum is easily flammable when combined with increased oxygen concentration via an oxygen cannula.
- Use one tube per person. Do not share the same tube, because of the risk of cross-contamination.
- Do not use if the packaging is damaged on receipt of the product.
- Do not use after the expiry date.
- Do not use on children under 6 years old.

Special warnings and precautions

- Do not use for more than 30 consecutive days.
- Consult your doctor if your symptoms do not improve, or if new symptoms develop.
- After nasal surgery, consult your doctor before use.
- Keep out of reach of children and pets.

Side effects

- In very rare cases pneumonia can occur after prolonged use of petrolatum (an ingredient of RhinoBlock) in the nose, according to the literature. It is therefore advisable to consult a doctor in the event of lung complaints during or after the use of RhinoBlock.
- Serious incidents related to use of this product must be reported to the manufacturer and to the competent authorities in the user's location or member state.

Storage

- The expiry date is stated on the packaging.
- Store for up to 12 months after opening.
- Store between 4°C and 30°C.

Date of latest revision of these instructions for use

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